



270 Washington Street  
Mount Vernon, NY 10553  
Ph: 855-237-7622  
Fax: 845-205-6005

## PROTEKT® BEVELED FLOOR MAT

REORDER # 51001

### DESIGN

The Protekt beveled floor mat reduces the impact of a fall from bed. This beveled floor mat is designed with a low profile of less than 0.7" (1.8cm) making it the **lowest floor mat on the market!** The tapered edge allows for easy equipment access, especially wheelchairs, and lessens the possibility of stumbling on the perimeter.

The lightweight design allows easy repositioning with minimal effort by all caregivers. The non-skid bottom ensures that this mat will always be in the right position.

### INSTRUCTIONS

Place the Protekt Beveled Floor Mat on the floor alongside the bed. The Protekt mat should always be used with the logo face up. When moving equipment across the mat ensure that the wheels are not locked as "dragging wheels" may damage the mat. Avoid sharp materials from contacting the mat. Never leave heavy materials on the mat for an extended amount of time and they may cause a permanent indentation. When unpacking, if the floor mat has been rolled up for shipping, some time may be needed for the mat to straighten itself.

### WARNINGS

Proactive Medical makes no guarantee, express or implied, that utilizing the Protekt Mat will prevent trauma to the resident or patient. The Protekt Mat is not a replacement for routine visual monitoring and is to be used in conjunction with a comprehensive fall management plan. It is the caregiver's responsibility to assess individual patient needs and to determine if this product meets that patients' needs.

### CLEANING AND INFECTION CONTROL

Clean mat with a damp cloth and spray and wipe cleaner. Do not use bleach-based or other caustic disinfectants as they may discolor the mat and damage the non-skid bottom. Ensure that the mat is completely dry before returning to service. The foam used has an antibacterial design to resist mold, mildew, stains, and odors.

### WARRANTY

5 Year non-prorated against manufacturer defects. Cuts, tears, wrinkles, marks and indentation are not covered by warranty. The non-skid bottom is designed with a material that will wear away over time. Standard wear of this material is not covered by the warranty and it does not affect the impact absorbing qualities of the mat.