



270 Washington Street  
Mount Vernon, NY 10553  
Ph: 855-237-7622  
Fax: 845-205-6005

ITEM # 10161

## **Protekt® Door Banner Alarm - instructions**

The Proactive Medical Protekt® Door Banner Alarm is a staff alert device used to signal that a door has been opened and a resident is about to leave the room. When the resident attempts to leave the room, the magnet is pulled from the unit and activates the alarm. The banner and alarm can be attached to the doorframe with the hook-and-loop closures. A more permanent installation option is available by installing a screw into the metal grommet on the side of the banner and attaching the alarm mounting bracket to the wall. The alarm unit requires (2) AAA batteries and has two volume settings; low and high.

### **SETUP**

1. Install the Protekt® Door Banner Alarm approximately 40" from floor level. Measure equal distances from the floor up on both sides of the door, and mark the locations.
2. **Alarm Side:** Peel backing from the loop closure attached to the backside of the alarm. Press firmly to the handle side of the doorframe at the location marked in step 1. For a permanent placement, fasten the bracket to a wall with a screw. Slide the alarm into the bracket.  
**Fixed Side:** Peel backing from hook closure and apply to doorframe at opposite mounting location. This side can also be mounted by fastening the loop piece to the wall through the metal grommet. Use a #8 or #10 screw to secure this side to the wall.
3. Attach door banner by fastening the hook-and-loop closure on the other side of the door, and attaching magnetic sensor to the alarm.
4. Mount banner so that STOP sign is visible to the resident attempting to exit the room. You may need to remove banner from adjustable strap and reverse it so that the STOP sign is visible to the resident.
5. Adjust the width of the banner, as necessary, to accommodate a wider or narrower doorway. The Protekt® Door Banner Alarm can be used in doorways up to 42" wide. Slide yellow banner to one side to reveal the adjustable strap. Reposition closure and reattach to the doorway.

### **BATTERY INSTALLATION**

1. Unit uses two AAA size Alkaline batteries, included.
2. To insert batteries:
  - Remove the two screws on the alarm's battery door. Take special care not to remove the rubber seal.
  - Insert batteries. When inserting batteries, observe the battery type and polarity (+/-) marking on the unit. Insert batteries accordingly.
  - Make sure that the rubber seal is properly positioned. Replace battery door by securing it with the two screws. **Do Not** over tighten the screws.