

Item#: PMCART

USER'S INSTRUCTION – OXYGEN CYLINDER CART (for E / D cylinders)

-  **WARNING:** If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts.
-  **WARNING! Important! Read and understand these instructions before assembling or using the oxygen cylinder cart. If you do not understand any part of these instructions, contact a healthcare professional for direction in the use of this product. If the oxygen cylinder cart is not properly assembled and adjusted, personal injury and/or damage could result.**
-  **WARNING:** Proactive Medical Products assumes no responsibility for any damage or injury caused by improper assembly or use of this product.


ASSEMBLY INSTRUCTIONS:



Handle



Base

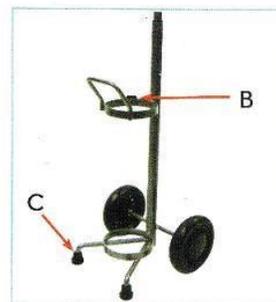

1. Insert the handle (A) into the base by pressing the pins.
2. You can adjust the knob on the base (B) to secure the tank.
3. If the tank seems unbalanced, check to see if tips (C) are pushed up all the way.

 **WARNING:** Ensure that the oxygen cylinder cart is assembled as described, and that all components are securely attached before use.

LIMITED WARRANTY:

This Proactive Medical Products oxygen cylinder cart is warranted to be free of defects in material and workmanship for one year of the original purchaser.

This device was built to exact standards and carefully inspected prior to shipment. This Limited One Year Warranty is an expression of our confidence in the material and workmanship of our products and our assurance to our customers for years of dependable services.

In the event of a defect covered by this warranty, Proactive Medical Products will, at Proactive's option, repair or replace the device. The warranty provides only for replacement of defective parts and does not cover shipping and labor charges. Proactive's sole obligation and your exclusive remedy under this warranty are limited to such repair and/or replacement.

This warranty does not cover device failure due to the user misuse, abuse, negligence, or normal wear and tear. This warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and require periodic replacement.

This warranty shall not apply to serial numbered devices if the serial number has been removed, altered, defaced; or to defect or damages caused by unauthorized parts, attachments, repairs, negligence, operated by incorrect voltages, accident, and circumstance beyond Proactive's control, or improper operation, maintenance or storage.

If you have a question regarding your Proactive Medical Products device or this warranty, please contact an authorized Proactive dealer.